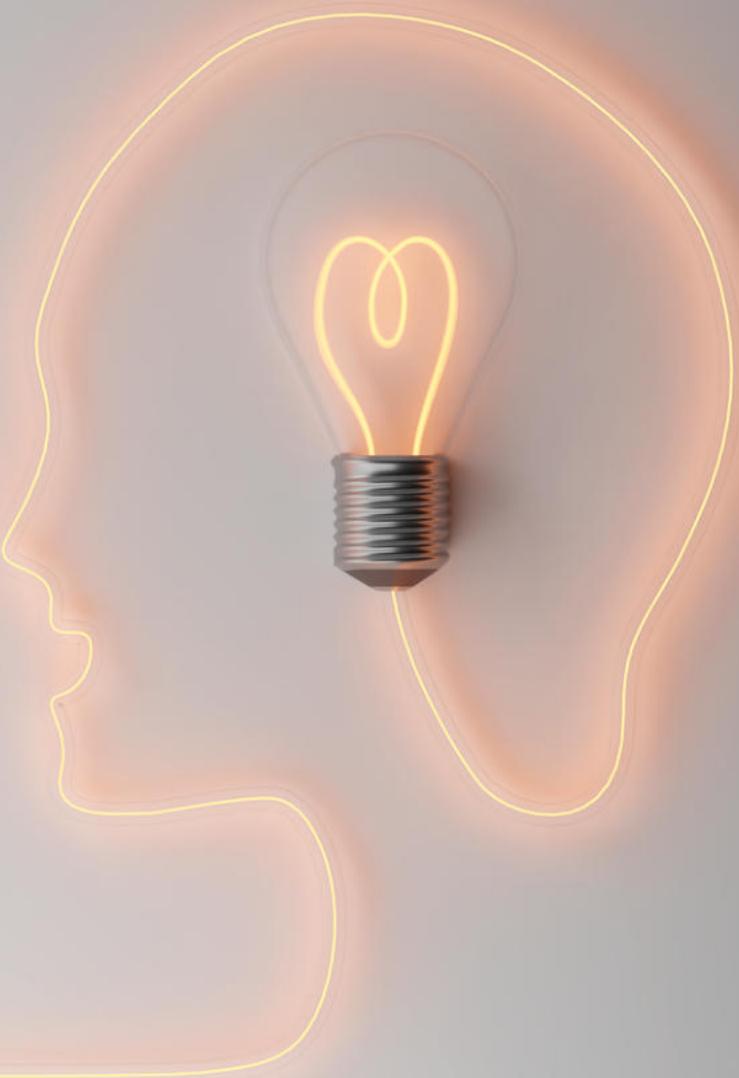
# NeuGlow 护智光

Powered by Delta M+ Technology

NeuGlow护智光 ——

**Illuminate  
Wisdom**

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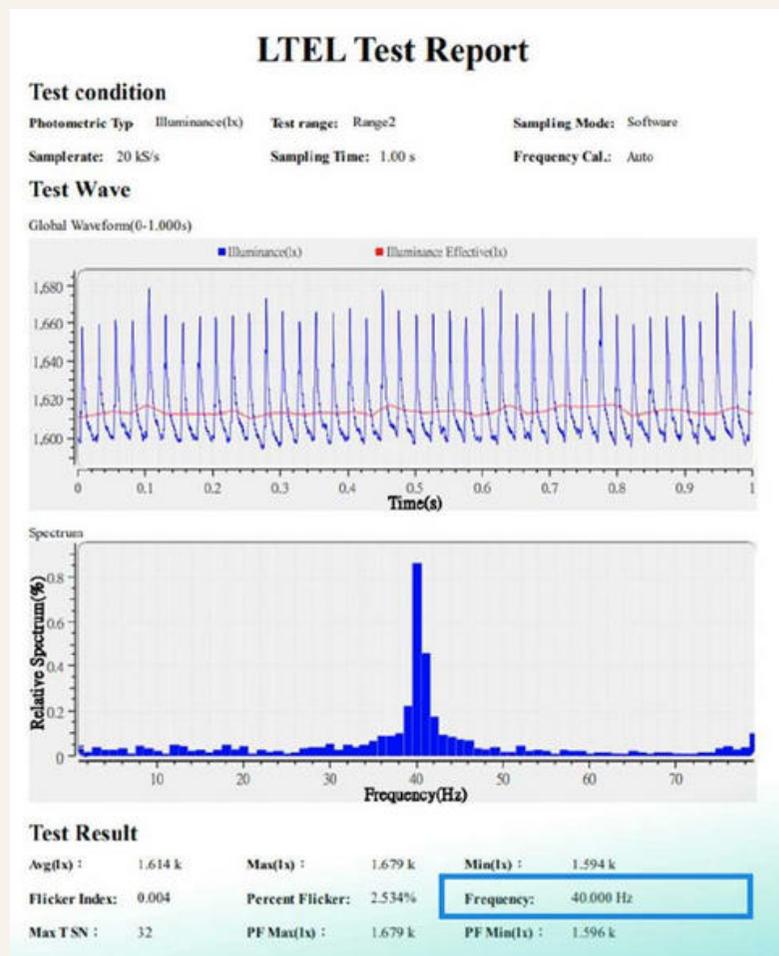
Journal of Alzheimer's Disease 90 (2022) 917–928  
DOI 10.3233/JAD-220307  
IOS Press

917

## Gamma Frequency Inhibits the Secretion and Aggregation of Amyloid- $\beta$ and Decreases the Phosphorylation of mTOR and Tau Proteins in vitro

Yuan-Han Yang<sup>a,b,c,d,1</sup>, Sun-Wung Hsieh<sup>a,c,e,f,1</sup>, Hsi-Wen Chang<sup>e</sup>, Jia-Li Sung<sup>b</sup>, Chih-Pin Chuu<sup>1</sup>, Chen-Wen Yen<sup>b,c,h,j,\*</sup> and Tzyh-Chyuan Hour<sup>c,g,k,\*</sup>

<sup>a</sup>Department of Neurology, Kaohsiung Medical University Hospital, Kaohsiung Medical University, Kaohsiung, Taiwan

<sup>b</sup>School of Post-Baccalaureate Medicine, College of Medicine, Kaohsiung Medical University, Kaohsiung, Taiwan

<sup>c</sup>Neuroscience Research Center, Kaohsiung Medical University, Kaohsiung, Taiwan

Journal of Neurology  
<https://doi.org/10.1007/s00415-024-12486-y>

ORIGINAL COMMUNICATION

## Exploring the impact of 40 Hz multi-luminaire light exposure in Alzheimer's dementia: insights from a convenient sampling, non-randomized case–control study

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Received: 21 March 2024 / Revised: 23 May 2024 / Accepted: 27 May 2024  
© Springer-Verlag GmbH Germany, part of Springer Nature 2024

### Abstract

**Background** Recent studies propose 40 Hz neural activity induction as a promising approach for managing Alzheimer's dementia (AD). However, traditional flickering light is suboptimal in addressing cognitive and neuropsychiatric symptoms (NPS) of AD. This study aims to investigate the clinical efficacy of a novel multi-luminaire lighting technology, with reduced perceptible flickering, for treating AD NPS.

**Methods** This study is a prospective, convenient sampling, non-randomized case–control investigation involving seventy-eight clinically diagnosed AD patients from 7 daycare centers. Thirty-five were exposed to 40 Hz light through Delta M+ BrainCare Light (M+), 4 h daily, 5 days/week, for 12 weeks. The other 43 patients served as controls. Sum of boxes of the Clinical Dementia Rating (CDR-SB) scale, Neuropsychiatric Inventory (NPI), and Zarit Burden Interview (ZBI) were assessed at baseline and the 13th week.

**Results** At baseline, the cases had worse cognitive function, lower cognitive score (Mini-Mental State Examination,  $p=0.04$ ; Cognitive Abilities Screening Instrument,  $p=0.04$ ), and advanced caregiver burden with higher ZBI scores ( $p<0.01$ ) than the controls. After the intervention, the cases had significant improvements in NPS as assessed using the NPI ( $p=0.02$ ), especially depression and euphoria symptoms ( $p=0.04$  and  $<0.01$ , respectively) and less caregiver burden (ZBI score,  $p<0.01$ ). In global function, the control group showed a significant decline in CDR-SB score ( $p<0.01$ ), while the cases did not.

**Conclusions** Results suggest M+ may slow global function decline, preserve cognitive function, improve NPS, and reduce caregiver burden in AD patients. Larger studies with biomarkers are needed to explore underlying mechanisms.

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